**LATERAL TRUNK
SUPPORT SYSTEM (LTSS)**

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Swing-Away Arm
with Large Pad
*Bras oscillant/avec
grand coussin*
NMAPS-DLTSS

LATERAL TRUNK SUPPORTS - COMPATABILITY MATRIX

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Note: In order to fit the lateral support arm on back supports where the cover interferes, it may be necessary to cut a slit in the cover for the arm to protrude through; or, fit the cover flap on the inside of the back shell to allow the lateral arm to pass.

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Only 2 screws are needed to attach a lateral support arm to the back shell. Although 3 slots are provided in the lateral support arm for mounting, any 2 may be used. Pass a screw through the back shell slot from the rear and pass through an appropriate slot in the lateral arm. Install a lock nut on the screw but do not tighten. Repeat with a second screw and nut, again do not tighten. Position the lateral as required and tighten both screws. The lock nut has a protrusion which fits in the arm slot to prevent it from turning.

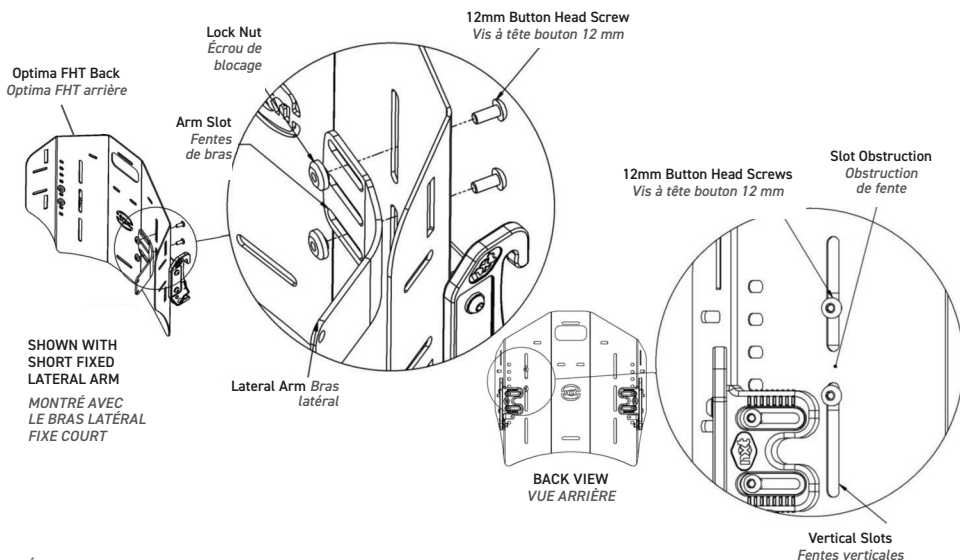
If, during lateral positioning, it is necessary to slide the lateral arm passed a slot obstruction remove only the screw and nut at the obstruction. Move the lateral arm and reinstall the screw and nut when a slot appears past the obstruction. If required the other screw can be removed allowing the lateral arm to move further. When a slot becomes visible reinstall the screw and nut and tighten both screws.

With slots in the back shell and lateral arm, the height and width of the lateral can be adjusted.

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

Only 2 screws are needed to attach a lateral support arm to the back shell. Although 3 slots are provided in the lateral support arm for mounting, any 2 may be used. Pass a screw through the back shell slot from the rear. Install the screw into a slot in the lateral. Install a lock nut on the screw but do not tighten. Install a second screw and nut, again do not tighten. Position the lateral as required and tighten both screws.

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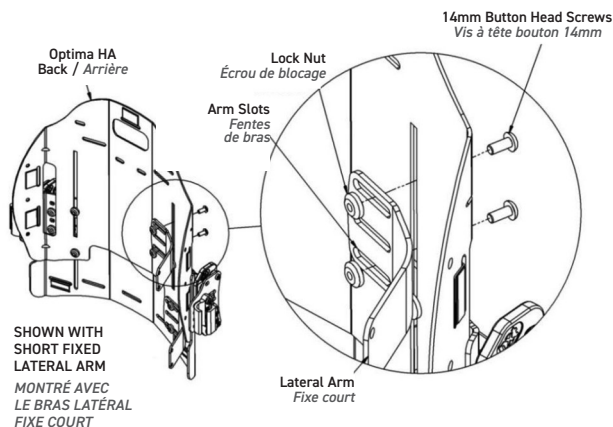If, during lateral positioning, it is necessary to slide the lateral arm passed a slot obstruction remove only the screw and nut at the obstruction. Move the lateral arm and reinstall the screw and nut when a slot appears past the obstruction. If required the other screw and nut can be removed allowing the lateral arm to move further. When a slot becomes visible reinstalled the screw and nut. Tighten both screws.

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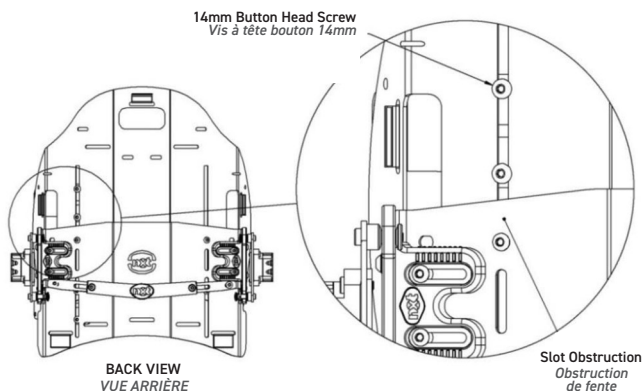
It may be necessary to remove the back height adjusting screw and lock nut for adjusting lateral positions. This is acceptable since the screw and nut for the lateral will replace these components.

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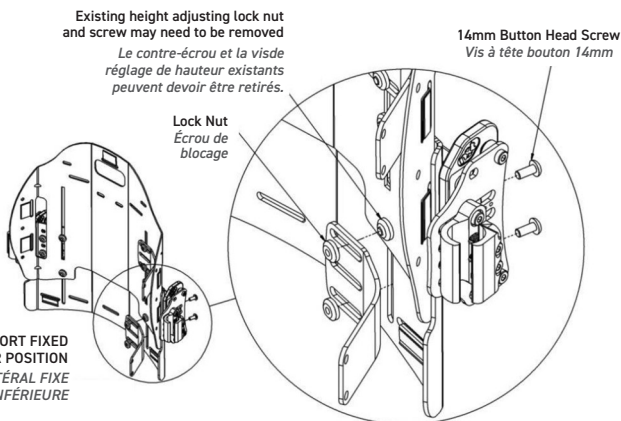TRUNK SUPPORT PADS / COUSSINS DE SUPPORT DE TRONC

Support pads are available in 2 sizes small (85mm x 75mm) and large (100mm x 85mm). The pad is attached to a support bracket using 2 button head screws. As shown in Figure 1 slots in the bracket allow the pad to be adjusted along the bracket.

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
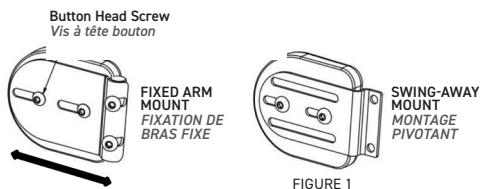
FIGURE 1

The bracket attaches the pad to the various lateral support arms available with the LTSS. Although several types of support arms are available, they share common features related to mounting and pad adjustment. Figure 2 shows an adjustable length swing-away lateral arm with dual pads attached. Adjustments available with this configuration are:

- Height on Back Shell – screws A (see section Lateral Support Arm Installation).
- Length of support arm – loosening screws B allow the support arm to be made longer or shorter.
- Angle and position of pads relative to support arm – loosening screws C allows the pads to be angled relative to the support arm and moved relative to the swing-away pivot point. With dual pads, the angle of each pad can be adjusted independently by loosening 1 screw at a time allowing the angle between pads to be changed. As mentioned above, the position of the pad on its support bracket can be adjusted by loosening screws D.

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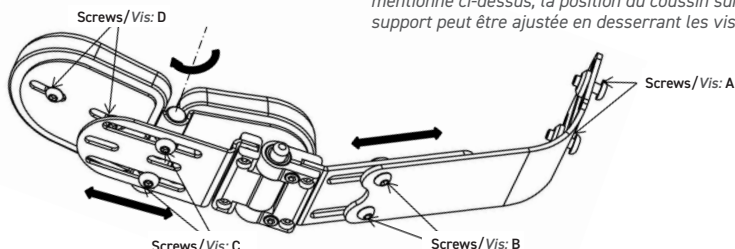
The Lateral Trunk Support Swing-Away unit is designed to allow the trunk support pad to be quickly and easily moved "out of the way" for transfers to and from a mobility or seating product. The support can then be returned accurately to its initial position when support is needed. The Swing-Away Hinge is released by pressing the Release Button, Figure 1, allowing the pad to be moved. Returning the pad to its initial position will allow the hinge lock to "snap" into the locked position holding the pad in place.

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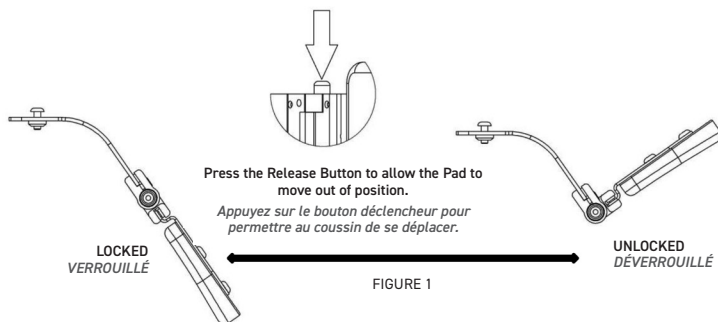
Two screws are loosened, Figure 2, to adjust the initial position of the support pad relative to the trunk support arm. First ensure the Swing-Away Hinge is in the locked position before adjusting the pad position. This can be determined by attempting to move the pad or by observing the position of the Release Button, it should be in the up position. Once the hinge is locked adjust the pad position by loosening the two screws so the hinge can rotate and the support pad move. Move the pad to its required position and tighten the two screws. The screws should be tightened firmly to prevent accidental movement of the pad during use.

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Make sure the Release Button is UP and the Swing-Away is LOCKED before loosening the angle adjust screws.

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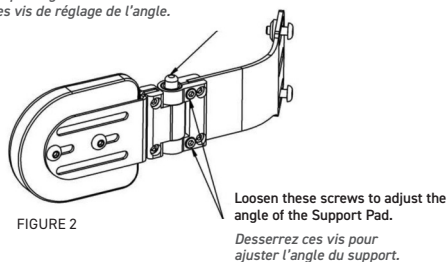SWING-AWAY ASSEMBLY – CONVERT LEFT TO RIGHT

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Figure 1 shows a swing-away lateral trunk support assembled for installation on the right side of an NXT back. To convert the swing-away to a left side installation, remove the 2 screws holding the swing-away hinge to the support arm. Turn the support arm over as it would be for a left side installation. Put the swing-away hinge onto the arm as shown in Figure 2. Reinstall the 2 screws and tighten. This procedure is used for fixed and adjustable length support arms.

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Note: Although the release button is usually on the top side of the swing-away hinge it will function if assembled on the bottom. This may be advantageous in some installations to prevent accidental pushing of the button. It is also preferable to keep the 2 screws used to adjust the angle of the pad on the outside of the assembly to allow easy access.

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Lateral Installation / Installation latérale

Note: Each lateral must be held to the back shell by 2 screws and slidingnuts. Three slots are provided in the attachment arms to allow 2 screws to be used if an obstruction is blocking one of the slots. If a bottom shell height adjustment screw is in the way it may be removed since the screws holding the lateral will perform the same function.

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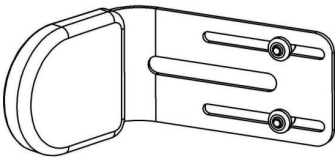
INSTALLATION AND ADJUSTMENT

1. Determine and assemble (if necessary) the appropriate lateral.
2. Determine the approximate location of the lateral.
3. From the rear of the back insert a screw so it passes through the shell lateral mounting slot and a lateral attachment arm slot.
4. Install a sliding nut on the front surface of the lateral attachment arm so it mates with the screw and the protrusion on the nut enters the slot.
5. Tighten this first screw to hold the lateral in place.
6. From the rear of the back insert the second screw through the shell lateral mounting slot and a lateral attachment arm slot which is accessible.
7. Install a second sliding nut on the front surface of the lateral attachment arm so it mates with the screw and the protrusion on the nut enters the slot.
8. Tighten the second screw.
9. It should now be possible by loosening (not removing) both screws to do final adjustment of the lateral.
10. Once the lateral is correctly positioned firmly tighten both screws.

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

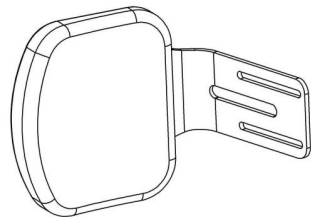
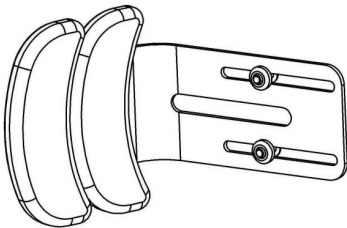
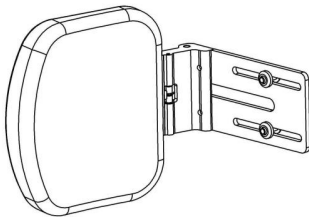
The large molded pad may be placed on its support arm in several positions, not limited to those shown below. It is important that 4 screws are always used to attach the pad. Pad positioning is the same on the Fixed or Swing-Away trunk supports.

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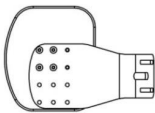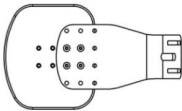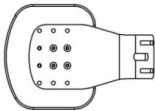
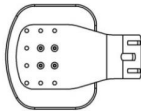
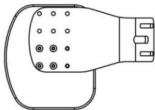
Note: ALWAYS use 4 SCREWS to attach pad.

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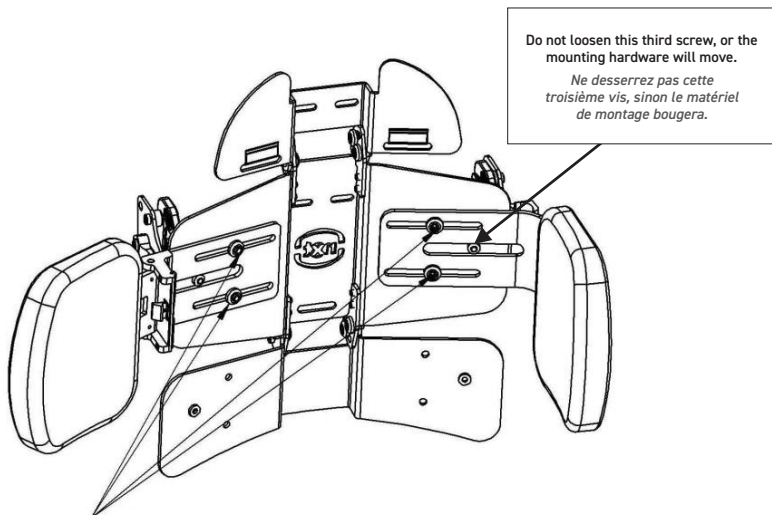
ATTACHING TRUNK SUPPORTS TO ARMADILLO BACK

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
Trunk Supports are attached to the middle section of the Armadillo using 2 fasteners each consisting of a screw and sliding nut. These screws thread into sliding nuts in slots on the laterals allowing the laterals to be infinitely adjustable between their most inner and outer positions only limited by the postural requirements of the occupant of the wheelchair and any structural components of the wheelchair. Loosening these screws allows the laterals to slide but will not affect the position of the mounting hardware.

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Each Lateral, whether Fixed or Swing-Away, must be held by 2 Fasteners to the Armadillo Shell as shown below.

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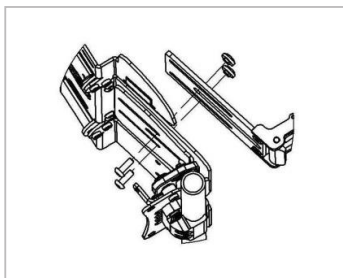Advena Limited | Tower Business Centre, 2nd Flr., Tower Street Swatar, BKR 4013 Malta