**NXT™ UNIVERSAL HEAD SUPPORT ADAPTER**  
*ADAPTATEUR DE SUPPORT DE TÊTE*  
*UNIVERSEL NXT™*

**INSTRUCTION MANUAL**  
*MANUEL D'INSTRUCTION*

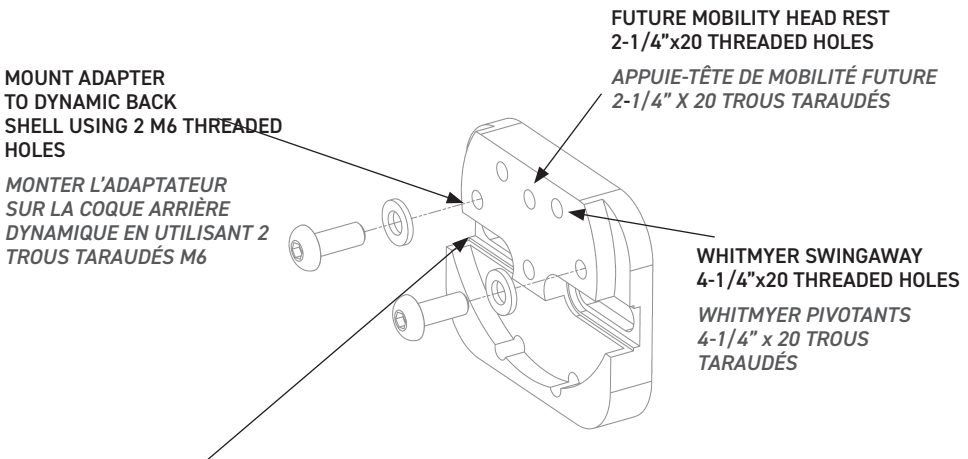
## NIB-1200 UNIVERSAL HEAD SUPPORT ADAPTER

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The Universal Head Support Adapter has several threaded holes and nut slots compatible with most common head support mounting brackets as shown in the following diagram.

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SLOTS FOR M6, 1/4" OR 5/16" NUTS WITH ADJUSTABLE SPACING FOR USE WITH HEAD SUPPORT USING 2 HORIZONTALLY SPACED SCREWS SUCH AS A OTTOBOCK, INVACARE, STEALTH AND WHITMYER REMOVABLE

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

# UNIVERSAL HEAD SUPPORT ADAPTER INSTALLATION

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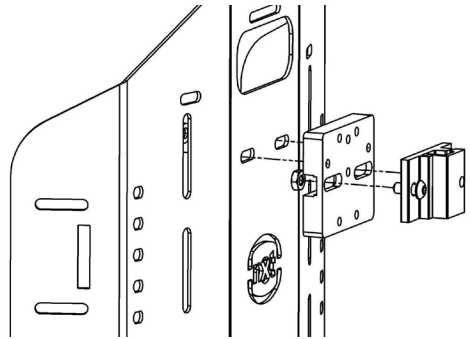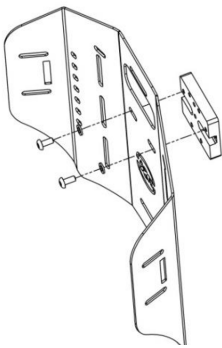The adapter is mounted to NXT series backs using 2 M6x14mm Button Head Cap Screws and M6 Flat Washer as shown below with an example of the installation of an OTTOBACK head support clamp.

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**Universal Head Support Adapter mounted to NXT series back with M6 screws and washers.**

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**OTTOBACK head support clamp mounted to the adapter with 1/4"-20 screws and nuts**

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The slotted holes and machined grooves allow the use of M6, 1/4" or 5/16" nuts with adjustable spacing to accommodate many different mounting arrangements. The nuts fit as shown below.

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Advena Limited | Tower Business Centre, 2nd Flr., Tower Street Swatar, BKR 4013 Malta